

diasend® Personal Uploader for t:slim X2™

Getting Started Guide



1 Log in or start a new account at www.diasend.com

The screenshot shows the diasend.com website on a monitor. A callout box highlights the login and registration section. The callout box contains the following elements:

- Username** and **Password** input fields, each with a question mark icon to its right.
- Log in** button.
- Forgot password?** link.
- Register here** button (highlighted in yellow).

Annotations with arrows point to these elements:

- An arrow points to the Username and Password fields with the text: "Already have an account? Log in with your user details."
- An arrow points to the Register here button with the text: "New to diasend? Create an account."

Note: Choose the icon for your country of residence.

2 Download the diasend Uploader software

The screenshot shows the registration completion page on a monitor. The page has a progress bar at the top with four steps: 1 Login details, 2 Personal information, 3 Share data, and 4 Confirm registration. The main content reads:

Thank you
Your diasend® Personal account has now been registered.
In order to upload your data to your diasend® Personal account, you will need the tool diasend® Uploader (free of charge) installed on your computer.


Below this text are two buttons for downloading the software: "diasend® Uploader for Windows" and "diasend® Uploader for OS X".

Annotations with arrows point to these buttons:


- An arrow points to the Windows button with the text: "Complete the registration for a new account and download the diasend® Uploader software in the final step of the registration process."
- Another arrow points to the OS X button.

Note: If you would like to install the diasend® Uploader, you can access the installation file from your account. To do so, log into your account and go to **Tools**.

3 Save the installation file



You have chosen to open:

 **DiasendUploaderPatient2.4.0_BuildR2e02.exe**


Which is a: binary file (16,2 MB)
From: <https://www.diasend.com>

Would you like to save this file?

Save File **Cancel**

*Note: Compatible operating systems:
Windows 7, 8 and 10, macOS X 10.6-10.13
Visit <https://goo.gl/YdGju1> to stay up to date with any compatibility changes.*

4 Locate the installation file and run the file

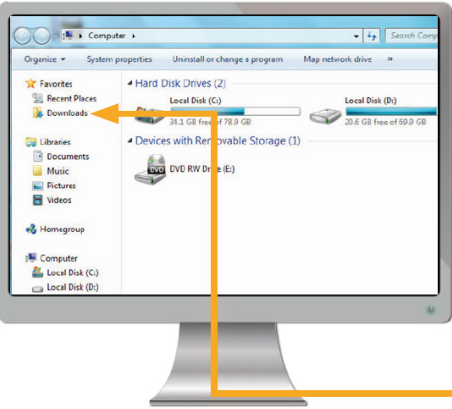


Depending on your web browser, you may need to search for the downloaded installation file, or you may just get a pop up window to run the file.

If you are a PC user: look in your recent downloads folder.

If you are a Mac user: look in the downloads folder in Finder.


5 Start diasend Uploader




Double click on the diasend Uploader icon on your desktop to start the program.

Note: Make sure you have restarted your computer before starting up diasend Uploader for the first time.

6 Connect your diabetes device cable to the computer




Connect the microUSB cable to the computer. Once the cable is connected, wait a few seconds for the cable drivers to be recognised, then connect to the t:slim X2 Insulin Pump.



Note: Make sure to use the cable that came with your t:slim X2 Insulin Pump.

7 Log in at www.diasend.com to view your data

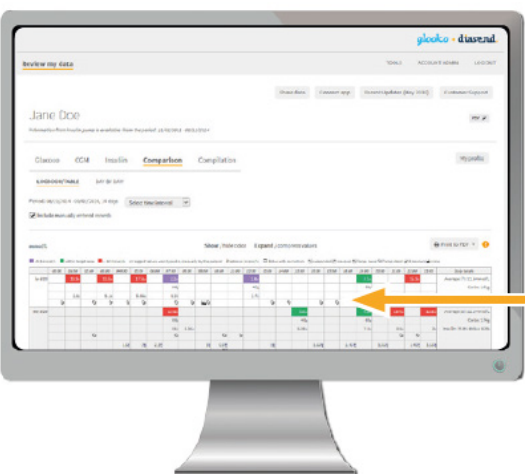


All your uploaded data is presented in easy-to-read tables and graphs.

The reports are divided into the following tabs:

Glucose CGM Insulin Comparison Compilation


8 Share your data with a clinic

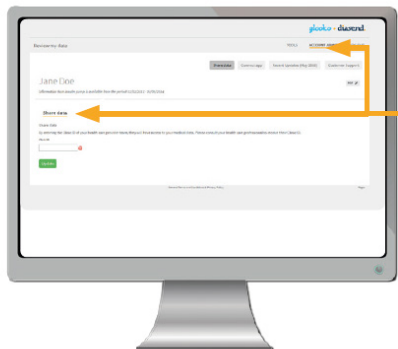

You can share your data with your healthcare provider by entering their clinic ID, under the tabs **Account admin** and **Share data**. Don't forget to click **Update**.

ACCOUNT ADMIN

Share data

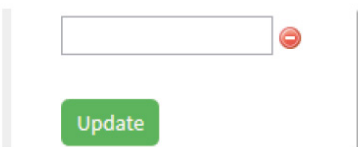

9 View your uploaded data in the diasend Mobile app



Download the app from Google Play or the App Store.
Log in with your personal diasend account details or create a new account.

With the diasend Mobile app you get:

- A summary of the latest uploaded data
- Easy to grasp reports on your insulin intake and glucose levels over time

For more information or for assistance, please contact us on
1300 851 056 or diabetes@amsl.com.au

amsldiabetes.com.au

